

Disinfecting Instruments

1. Please use a tub with lid to disinfect instruments.



2. Mix the instrument disinfectant solution precisely in accordance with the manufacturer's instructions while using a suitable measuring cup to do so.

Wear gloves and, if necessary, protective goggles.



3. Please note the required contact time!



4. Clean and rinse the instruments under running water after disinfecting them and then dry them. Keep disinfected and cleaned instruments in such a way that they are protected from dust (e.g. in a drawer).



Please note:

- You may also use a ready instrument disinfectant which doesn't require preparation (e.g. dissolved in water). Such disinfectants, however, are more expensive.
- Only tested instrument disinfectants should be used (e.g. list issued by the Verbund der angewandten Hygiene e.V. (VAH – Registered Association of Applied Hygiene)).